

# Faxitron<sup>®</sup> Core

Specimen Radiography System



## QUICK START GUIDE

### 1. POWER UP

- Ensure the power switch on the back of the Faxitron Core cabinet is set to the **ON** position
- Turn the **key** on the front of the Faxitron Core cabinet to **on**
- Turn the computer and monitor **on**

### 2. START SOFTWARE

- Verify there is nothing in the Faxitron Core sample drawer
- Double click the **Vision** icon in the center of the desktop

### 3. CALIBRATE

- You will be prompted to calibrate once you log in  
Press the **"Calibrate"** button to proceed

### 4. PATIENT DATA

- Click **"Start Procedure"**
- Click **"Select from Worklist"**
- Locate patient's name and double click it
- Enter any missing required fields such as Laterality
- Click **"Accept"**

### 5. X-RAY IMAGING

- Place specimen(s) in the Faxitron Core specimen tray
- Place the specimen tray in the drawer
- Close the Faxitron Core drawer
- Press the green **"X-ray"** button to activate x-rays - image will appear after a few seconds

### 6. IMAGE MANIPULATION

- Hold down the **left mouse button** and drag the mouse to change brightness and contrast for window/level function
- **Adjust zoom** to enlarge the image for viewing

### 7. STORING TO PACS

- Click the **"PACS Store"** button on left side of screen

### 8. PRINTING

- Click the **"PACS Print"** button on left side of screen

### 9. SHUT DOWN

- Click the **"System Shutdown"** button on the left side and then click **"OK"**
- Turn key on Faxitron Core cabinet to **off**

 [www.hologic.com/package-inserts](http://www.hologic.com/package-inserts)



**Hologic, Inc.**  
36 Apple Ridge Road  
Danbury, CT 06810 USA  
1.800.447.1856  
[www.hologic.com](http://www.hologic.com)



**Hologic, BV**  
Da Vincilaan 5  
1930 Zaventem  
Belgium  
Tel: +32 2 711 46 80  
Fax: +32 2 725 20 87

**AU SPONSOR**  
**Hologic Pty Ltd**  
Suite 302, Level 3  
2 Lyon Park Road  
Macquarie Park NSW 2113  
Tel: 02 9888 8000



*Faxitron Core system also known as CoreVision System.*

04-4046-00 Rev. 032 (8/21) Hologic Inc. ©2021 All rights reserved. Hologic, CoreVision, Faxitron and The Science of Sure are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and/or other countries. This information is intended for medical professionals in the U.S. and other markets and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshow, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your local Hologic representative.